



American MasterTech
scientific laboratory supplies

Carnoy's 2000™

Ready to Use Histological Tissue Fixative

(Rev. 3/8/18)

Carnoy's 2000™, The product and its purpose:

Carnoy's 2000™ is a Histological tissue fixative specially formulated without toxic and poisonous Chloroform that will fix tissue in the same manner as traditional Carnoy's Fluid. Carnoy's 2000™ is routinely used preserving RNA, nissl granules, and glycogen. Carnoy's 2000™ is intended for In Vitro Diagnostic use. Tissue fixed with Carnoy's 2000™ can be used for routine Special Stains and Immunohistochemical (IHC) stains. Carnoy's 2000™ is compatible with routine processing alcohols. Active ingredients include; Ethanol (<75%) and Acetic Acid (<30%).

Carnoy's 2000™, Instructions for use:

Use Carnoy's 2000™ as you would traditional Carnoy's Fluid. For RNA, nissl granules, and glycogen, increase fixation time 10 to 15% longer than traditional Carnoy's Fluid

Carnoy's 2000™ Storage and Stability:

Carnoy's 2000™ should be stored at 15-30 C and is stable under normal laboratory conditions. See product label for shelf life.













Carnoy's 2000™, Precautions:

See Safety Data Sheet (SDS) for warnings and precautions.

Carnoy's 2000™, Disposal:

Dispose Carnoy's 2000™ in accordance with all local, state, federal, environmental and/or EU regulations.

CE MARKINGS AND DESIGNATIONS:

	Catalogue Number		Temperature Limitation		Manufacturer	American MasterTech Scientific 1330 Thurman St. Lodi, CA 95240 USA Tel 800 860 4073 Fax 209 368 4136
	Batch Code		Use By		Representative	Emergo Europe Prinsessegracht 20 2514 AP The Hague The Netherlands
	In Vitro Diagnostic Medical Device		Consult Instructions Prior to Use			
	Flammable		Corrosive		Health Hazard	

Copyright © 2015-2018 American MasterTech Scientific. All rights reserved.

No part of these pages may be used for any purpose other than personal use. Therefore, reproduction, modification, storage in a retrieval system or retransmission, in any form or by any means, for reasons other than personal use, is strictly prohibited without prior written permission.