

## CD3, T Cell (CAL54)

Rabbit Monoclonal Antibody

SAM-QHD-RM151-2 mL (10 tests)

QHD-RM151-15 mL (100 tests)

Document Number: IFU-RM151-CD3, T Cell-05152018

Release Date: 05/15/2018, IFU-292 Rev A

Immunogen	Clone	Species	Isotype	Primary Antibody Diluent
Synthetic peptide derived from a region of the human CD3 protein	CAL54	Rabbit	IgG1	NA

Catalog Number	Description
SAM-QHD-RM151-2 mL	2 mL Ready To Use antibody for use with StatLab Quantum HD HRP or Quantum HD AP Detection kit on fully automated StatLab Quantum HDx Platform.
QHD-RM151-15 mL	15 mL Ready To Use antibody for use with StatLab Quantum HD HRP or Quantum HD AP Detection kit on fully automated StatLab Quantum HDx Platform

### Intended Use For In Vitro Diagnostic Use.

CD3 (CAL54) Rabbit Monoclonal Primary Antibody is intended for laboratory use in the detection of the CD3 protein in formalin-fixed, paraffin-embedded tissue stained in qualitative immunohistochemistry (IHC) testing.

The results using this product should be interpreted CD3 antibody is intended for qualified laboratories to qualitatively identify by light microscopy the presence of associated antigens in sections of formalin-fixed, paraffin-embedded tissue sections using IHC test methods.

### Summary and Explanation

CD3's immunohistochemical detection locates the cytoplasmic component of CD3 protein. Anti-CD3 is considered to be a pan-T-cell marker and reacts with an antigen present at the surface and in the cytoplasm of T lymphocytes. Anti-CD3 is widely used for the identification of immature and mature T-cell malignancies.<sup>1-8</sup>

### Materials and Methods Provided

The stated primary antibody product contains reagent in a vial made for use with the StatLab Quantum HDx IHC slide stainer. The vial is equipped with an RFID tag that is ready by the slide stainer to provide product and

lot specific information.

The predilute immunoglobulin concentration range for this product is 1.0-10.0 ug/ml. This antibody is diluted in Tris Buffer, pH 7.3-7.7, with 1% BSA and 0.1% Sodium Azide.

### Materials and Reagents Needed but Not Provided

The following reagents and materials may be required for staining but are not provided with the primary antibody. Please refer to our website at: [www.StatLab.com](http://www.StatLab.com).

1. Quantum HD HRP Detection Kit (Cat. No.: QHC-U-15-HRP-KIT)
2. Quantum HD Retrieval Solution, pH 9.0 (Cat. No.: QHD-003)
3. Quantum HD Retrieval Solution, pH 6.0 (Cat. No.: QHD-002)
4. Quantum HD DS2 (Cat. No.: QHD-007)
5. Quantum HD Block (Cat. No.: QHD-006)
6. Quantum HD AP Detection Kit (Cat. No.: QHC-U-15-HRP-KIT)
7. Wash Buffer (Cat. No.: QHD-015)
8. Positive and Negative Tissue controls

### Storage and Handling

Store at 2-8°C. Do NOT freeze. When stored properly, the reagents are stable to the date indicated on the label. The presence of an unusual odor or precipitate indicates that the antibody is deteriorating and should not be used.

### Principles of the Procedures

Antigen detection by immunohistochemistry (IHC) is a two-step process involving first, the binding of a primary antibody to the antigen of interest, and second, the detection of bound antibody by a chromogen. The primary antibody may be used in IHC using manual techniques or using automated IHC Staining Systems.

### Warnings and Precautions

1. This antibody contains less than 0.1% sodium azide. Concentrations less than 0.1% are not reportable as hazardous materials<sup>9</sup>.
2. Sodium azide (NaN<sub>3</sub>) used as a preservative is toxic if ingested. Sodium azide may react with lead and copper plumbing to form highly explosive metal azides. Upon disposal, flush with large volumes of water to prevent azide build-up in plumbing<sup>10</sup>.
3. Specimens, before and after fixation and all materials exposed to them, should be handled as if capable of transmitting infections and disposed of with proper precautions<sup>11</sup>.
4. Never pipette reagents by mouth and avoid contacting the skin and mucous membranes with reagents and specimens. If reagents or specimens come in contact with sensitive areas, wash with copious amounts of water.
5. Microbial contamination of reagents may result in an increase in nonspecific staining.
6. Incubation times or temperatures other than those specified may give erroneous results. The use must validate any such change.
7. The SDS is available upon request



8. Do not use reagents beyond the expiration date printed on the vial.
9. The user must validate any storage conditions other than these specified in the package insert

## Specimen Collection and Preparation

Tissues fixed in 10% formalin are suitable for use prior to paraffin embedding<sup>12,13</sup>.

The user is advised to validate the use of the products with their tissue specimens prepared and handled in accordance with their laboratory practices

### StatLab Quantum HDx Recommended Staining Procedure:

Instrument Parameters	HRP	AP
Detection Kit	QHC-U-15-HRP-Kit	QHC-U-15-AP-Kit
Retrieval Reagent	QHD-High pH	QHD-High pH
Antibody Incubation Time	10-45 minutes	10-45 minutes

## Troubleshooting

Positive and negative controls should be run simultaneously with all patient specimens. If unexpected staining is observed which cannot be explained by variations in laboratory procedures and a problem with the antibody is suspected, contact StatLab IHC Technical Support via Email at: [ihctech@statlab.com](mailto:ihctech@statlab.com) or call us at (800) 442-3573.

## Cellular Localization and Positive Tissue Control

Positive Tissue Control	
Tissue	Visualization
Tonsil	Cytoplasmic

## Limitations of the Procedure

IHC is a complex technique involving both histological and immunological detection methods. Tissue processing and handling prior to immunostaining can also cause inconsistent results. Variations in fixation and embedding or the inherent nature of the tissue may cause variations in results<sup>14</sup>. Endogenous peroxidase activity or pseudo peroxidase activity in erythrocytes and endogenous biotin may cause non-specific staining depending on detection system used. Tissues containing Hepatitis B Surface Antigen (HBsAg) may give false positive with horseradish peroxidase systems<sup>14</sup>. Improper counterstaining and mounting may compromise the interpretation of results.

## Performance Characteristics

The optimum antibody protocols for a specific application can vary. These include, but are not limited to: fixation, heat-retrieval method, incubation times, and tissue section thickness and detection kit used. Due to the superior sensitivity of these unique reagents, the recommended incubation times may vary. The data sheet recommendations and protocols are based on exclusive use of products manufactured for StatLab. Ultimately, it is the responsibility of the investigator to determine optimal conditions. These products are tools that can be used for interpretation of morphological

findings in conjunction with other diagnostic tests and pertinent clinical data by a qualified pathologist.

## NOTE

There are no expressed or implied warranties which extend beyond this datasheet. StatLab is not liable for personal injury, property damage or economic loss caused by this product

## References

1. Denning SM, et al. Activation of human thymocytes via CD3 and CD2 molecules. In:McMichael AJ, et al. eds. Leucocyte Typing III, White Cell Differentiation Antigens. Oxford New York-Tokyo. Oxford University Press. 1987; 144-7.
2. Beverley PC, et al. Distinctive functional characteristics of human "T" lymphocytes defined by E rosetting or a monoclonal anti-T cell antibody. Eur J Immunol. 1981; 11:329-34.
3. Clevers H, et al. The transmembrane orientation of the epsilon chain of the TcR/CD3 complex. Eur J Immunol. 1988; 18:705-10.
4. Campana D, et al. The cytoplasmic expression of CD3 antigens in normal and malignant cells of the T lymphoid lineage. J Immunol. 1987; 138:648-55.
5. Hedvat CV, et al. Application of tissue microarray technology to the study of non-Hodgkin's and Hodgkin's lymphoma. Hum Pathol. 2002; 33:968-74.
6. Karube K, et al. Non-B, non-T neoplasms with lymphoblast morphology: further clarification and classification. Am J Surg Pathol. 2003; 27:1366-74.
7. Dogan A, et al. Micronodular T-cell/histiocyte-rich large B-cell lymphoma of the spleen: histology, immunophenotype, and differential diagnosis. Am J Surg Pathol. 2003; 27:903-11.
8. Axordorph U, et al. T-cell-rich B-cell lymphoma - diagnostic and therapeutic aspects. APMIS. 2002; 110:379-90.
9. U.S. 29CFR 1910.1200, OSHA Hazard Communication and EC Directive 91/155/EC.
10. Center for Disease Control, 1976, National Institute of Occupational Safety and Health, 1976.
11. U.S. Department of Health and Human Services: Centers for Disease Control and Prevention. Guidelines for Safe Work Practices in Human and Animal Medical Diagnostic Laboratories, Supplement/Vol 61, January 6, 2012.
12. Kiernan, Microscopy Today 00-1 pp. 8-12, (2000)
13. Sheehan and Hrapchak, Theory and Practice of Histotechnology, Second Edition, Battelle Press, 1980
14. Nadji and Morales, AR Ann N.Y. Acad Sci 420:134-9, 1980
15. Omata M et al, Am J Clin Pathol 73(5): 626-32, 1980

