

Intended Use

Acrymount Plus is a rapid-drying, toluene-based mounting medium/liquid coverglass.

General Information

Compatible with xylene, and xylene-substitute clearing agents including d-Limonene. Slides mounted with Acrymount™ Plus may be screened in as little as 10 minutes. Formulated to reduce oxidation, preventing yellowing and fading in H&E, IHC and special stains after coverslipping. Maintains an optimal refractive index for viewing and photo microscopy.

For labs seeking to eliminate final clearing with xylene, tissue may be coverslipped directly from absolute Isopropyl Alcohol.

Compatible with both manual and most automated coverslipping processes.

Storage and Stability

Acrymount Plus can be stored at ambient temperature, and is stable to the date indicated on the label when stored properly. Product is hazardous and should be stored in a way consistent with local and state guidelines.

Packaging

SL80Plus-4 – 4, 4 oz squeeze bottles

SL80Plus-16 – 1, 16 oz container

Please contact tech@StatLab.com with any additional questions.