

Instructions for Use

Scored StatPrep Cervical Brush



CATALOG NUMBER	DESCRIPTION	UNIT OF MEASUREMENT
CB500-100	Scored StatPrep Cervical Brush, 8" sterile	100 per package

INTENDED USE

Cervical brushes are scored brushes intended for use in liquid based cytology endocervical cell collection. They are designed specifically to gently collect an adequate amount of endocervical material for a more accurate pap smear diagnosis. Cervical brushes are sterile and individually packaged.

APPLICATIONS

Endocervical cell collection in pap smear, chlamydia and HPV testing.

STORAGE AND STABILITY

Store in a cool dry place

PROCEDURE

- 1 Insert brush into the cervix until only the bottom-most fibers are exposed.
- 2 Slowly rotate $\frac{1}{4}$ or $\frac{1}{2}$ turn in one direction.
*Do not over-rotate the brush
- 3 Rinse the brush in StatPrep Preserve-Gyn by rotating device in the solution while pushing against the wall of the container, then swirl brush to release any other material.
- 4 Discard brush.

Please contact tech@statlab.com with any additional questions.



StatLab Medical Products

2090 Commerce Drive, McKinney, TX 75069

StatLab.com Phone: +1 800-442-3573

