Human Spleen

Acid Fast Bacteria (A.F.B.) Stain Kit Procedure

100ml Kit Item #: KTAFB

Liter Kit Item#: KTAFBLT

Pint Kit Item #: KTAFBPT

Gallon Kit Item#: N/A

Control Slide(s)	Item#
Acid Fast Bacteria	CSA0125P

Included Components
Advanced Carbol Fuchsin™
1% Acid Alcohol
Light Green Counterstain

PRINCIPLE AND RESULTS: This kit is intended for use by laboratory professionals to stain routinely prepared paraffin embedded tissue specimens (in vitro) to identify Acid Fast Bacteria (mycobacteria), including *Mycobacterium tuberculosis*. All Acid Fast Bacteria stain bright red and other tissue light green.

SPECIMEN CRITERIA: Appropriately fixed, paraffin-embedded, 6µm tissue section.

STORAGE AND USAGE NOTES: Filter Advanced Carbol Fuchsin™ when a shiny gold layer appears on the surface. Store/Use each component according to the temperature and expiration on the label. To demonstrate *Mycobacterium leprae* and *Nocardia*, use American MasterTech's Fite's Stain Kit (Item#: KTFIT).

PRECAUTIONS: For use by laboratory professionals. See SDS for complete warnings, precautions, hazard and precautionary statements, and disposal information.

STAINING PROCEDURE:

 Color coordinated steps denote stain baths that can be reused during autostainer configuration.

#	Action	With	Heat °C	Time		Details
				Mins	Secs	
1	Deparaffinize	Xylene or Substitute, 2 changes	--	5	--	5 minutes each change or as required if using a xylene substitute.
2	Rinse	Absolute Alcohol, 3 changes	--	1	--	1 minute each change or as required if using graded alcohols.
3	Rinse	Running Tap Water	--	1	--	
4	Immerse	Advanced Carbol Fuchsin™	--	10-15	--	Once complete, rinse in running tap water (1 minute) and continue.
5	Immerse	1% Acid Alcohol	--	--	3-5	Differentiate with agitation until background is pale pink. Use a quick tap water rinse and microscope to check for correct differentiation. Once complete, rinse in running tap water (1 minute) and continue.
6	Immerse	Light Green Counterstain	--	1	--	Without rinsing, continue to next step.
7	Dehydrate	Absolute Alcohol, 3 changes	--	1	--	1 minute each change.
8	Clear	Xylene or Substitute, 3 changes	--	1	--	1 minute each change or as required if using a xylene substitute.
9	Coverslip	Permanent Mounting Media	--	--	--	

1. A.F.I.P. Manual, pg. 205; Mallory Pathological Technique, pg. 275
2. With modifications by AMTS R&D Department, 1979-2019.

AUTOSTAINER CONFIGURATION AND NOTES:

This stain kit in the pint or larger size may be easily adapted for use on most open-platform autostainers using the staining procedure grid on the reverse side of this page. A minimum of 4 baths is required to perform this procedure excluding deparaffinization, hydration, dehydration, and clearing, or 15 baths to run the complete procedure.

TEST YIELD: *Assumes pint kit and maximum slides per run. Actual Results may vary. S.C. denotes number of slides between "Solution Change".

Bath Type	Uses	Slides	S.C.	Bath Type	Uses	Slides	S.C.
20ml Plastic Slide Jar	25	500	20	250ml Glass Stain Dish	2	375	188
30ml Glass Coplin Jar	16	480	30	200ml Bath Autostainer	2	320	160
40ml Hellendahl Jar	12	480	40	400ml Bath Autostainer	1	340	340

CE MARKINGS AND DESIGNATIONS:

REF	Catalogue Number		Temperature Limitation		Manufacturer	American MasterTech Scientific 1330 Thurman St. Lodi, CA 95240 USA Tel 800 860 4073 Fax 209 368 4136
LOT	Batch Code		Use By	EC REP	Representative	Emergo Europe Prinsessegracht 20 2514 AP The Hague The Netherlands
IVD	In Vitro Diagnostic Medical Device		Consult Instructions Prior to Use	CE		
	Flammable		Corrosive		Health Hazard	

CONTACT INFORMATION:

American MasterTech Scientific
1330 Thurman St.
Lodi, CA 95240 USA
Tel 800 860 4073
Fax 209 368 4136
www.americanmastertech.com

StatLab
2090 Commerce Drive
McKinney, TX 75069 USA
Tel 800 442 3573
Fax 972 436 1369
www.statlab.com

1. A.F.I.P. Manual, pg. 205; Mallory Pathological Technique, pg. 275
2. With modifications by AMTS R&D Department, 1979-2019.

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COMPOSANTS INCLUS: Advanced Carbol Fuchsin™, 1% Acid Alcohol, Light Green Counterstain

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COMPONENTES INCLUIDOS: Advanced Carbol Fuchsin™, 1% Acid Alcohol, Light Green Counterstain

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