Human Liver

Alkaline Congo Red Amyloid Stain Kit Procedure

100ml Kit Item #: KTACR

Liter Kit Item#: N/A

Pint Kit Item #: KTACRPT

Gallon Kit Item#: N/A

Control Slide(s)	Item#
Amyloid Alpha	CSA0425P

Included Components	
Alkaline Congo Red Solution	Alkaline Alcohol Solution
Modified Mayer's Hematoxylin	1% Sodium Hydroxide

PRINCIPLE AND RESULTS: This kit is intended for use by laboratory professionals to stain routinely prepared paraffin embedded tissue specimens (in vitro) or frozen sections to identify amyloid deposits. Amyloid stain pale pink to red (Birefringent if viewed using a polarized light microscope) and nuclei blue.

SPECIMEN CRITERIA: Appropriately fixed, paraffin-embedded or frozen tissue section cut at 7-9µm.

STORAGE AND USAGE NOTES: Store/Use each component according to the temperature and expiration on the label.

PRECAUTIONS: For use by laboratory professionals. See SDS for complete warnings, precautions, hazard and precautionary statements, and disposal information.

WORKING ALKALINE CONGO RED SOLUTION PREPARATION: Prepare solution at time of use.

#	Action	Amount	Chemical/Reagent	Details
1	Add	50ml	Alkaline Congo Red Solution	Into a coplin jar or container.
2	Add	8 Drops	1% Sodium Hydroxide	Mix thoroughly. If allowed to stand, mix before use.

STAINING PROCEDURE:

 Color coordinated steps denote stain baths that can be reused during autostainer configuration.

#	Action	With	Heat °C	Time		Details
				Mins	Secs	
1	Deparaffinize	Xylene or Substitute, 2 changes	--	5	--	5 minutes each change or as required if using a xylene substitute.
2	Rinse	Absolute Alcohol, 3 changes	--	1	--	1 minute each change or as required if using graded alcohols.
3	Rinse	Running DI Water	--	1	--	
4	Immerse	Working Alkaline Congo Red Solution	--	10	--	Once complete, rinse in running DI water (30 seconds) and continue.
5	Immerse	Alkaline Alcohol Solution	--	--	1	5-10 dips for 1 second each until Amyloid is pink to red. Use a quick DI water rinse and microscope to check for correct differentiation. Once complete, rinse in running DI water (1 minute) and continue.
6	Immerse	Modified Mayer's Hematoxylin	--	1	--	
7	Rinse	Running Tap Water	--	1	--	
8	Dehydrate	Absolute Alcohol, 3 changes	--	1	--	1 minute each change.
9	Clear	Xylene or Substitute, 3 changes	--	1	--	1 minute each change or as required if using a xylene substitute.
10	Coverslip	Permanent Mounting Media	--	--	--	

- Carson FL Hladik C: Histotechnology A Self-Instructional Text; 2009, 152 – 154.
- With modifications by AMTS R&D Department, 1979-2019.

AUTOSTAINER CONFIGURATION AND NOTES:

This stain kit in the pint size may be easily adapted for use on most open-platform autostainers using the staining procedure grid on the reverse side of this page. Standard Working Alkaline Congo Red Solution preparation procedures yields approximately 50ml of solution and must be scaled up to accommodate desired autostainer bath size. A minimum of 5 baths is required to perform this procedure excluding deparaffinization, hydration, dehydration, and clearing, or 16 baths to run the complete procedure.

TEST YIELD: *Assumes pint kit and maximum slides per run. Actual Results may vary. S.C. denotes number of slides between "Solution Change".

Bath Type	Uses	Slides	S.C.	Bath Type	Uses	Slides	S.C.
20ml Plastic Slide Jar	25	225	9	250ml Glass Stain Dish	2	169	85
30ml Glass Coplin Jar	16	216	14	200ml Bath Autostainer	2	144	72
40ml Hellendahl Jar	12	216	18	400ml Bath Autostainer	1	153	153

CE MARKINGS AND DESIGNATIONS:

REF	Catalogue Number		Temperature Limitation		Manufacturer	American MasterTech Scientific 1330 Thurman St. Lodi, CA 95240 USA Tel 800 860 4073 Fax 209 368 4136
LOT	Batch Code		Use By	EC REP	Representative	Emergo Europe Prinsessegracht 20 2514 AP The Hague The Netherlands
IVD	In Vitro Diagnostic Medical Device		Consult Instructions Prior to Use	CE		
	Flammable		Irritant		Health Hazard	
GHS02		GHS07		GHS08		

CONTACT INFORMATION:

American MasterTech Scientific
1330 Thurman St.
Lodi, CA 95240 USA
Tel 800 860 4073
Fax 209 368 4136
www.americanmastertech.com

StatLab
2090 Commerce Drive
McKinney, TX 75069 USA
Tel 800 442 3573
Fax 972 436 1369
www.statlab.com

1. Carson FL Hladik C: Histotechnology A Self-Instructional Text; 2009, 152 – 154.
2. With modifications by AMTS R&D Department, 1979-2019.

-
1. Carson FL Hladik C: Histotechnology A Self-Instructional Text; 2009, 152 – 154.
 2. With modifications by AMTS R&D Department, 1979-2019.

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COMPOSANTS INCLUS: Alkaline Congo Red Solution, Alkaline Alcohol Solution, Modified Mayer's Hematoxylin, 1% Sodium Hydroxide

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