Human Gastric Tissue

## CellMate™ Stain Kit Procedure

<b>100ml Kit Item #:</b> KTCEL	<b>Liter Kit Item#:</b> N/A
<b>Pint Kit Item #:</b> KTCELPT	<b>Gallon Kit Item#:</b> N/A

Control Slide(s)	Item#	Included Components
Bone Marrow	CSB0325P	Cellmate-A™
Helicobacter Pylori	CSH0125P	Cellmate-B™
Mast Cell	CSM0125P	

**PRINCIPLE AND RESULTS:** This kit is intended for use by laboratory professionals to stain routinely prepared paraffin embedded tissue specimens (in vitro) to identify Helicobacter pylori, bone marrow, and mast cells. Helicobacter pylori stain blue, mast cells purple, and bone marrow metachromatic.

**SPECIMEN CRITERIA:** Appropriately fixed, paraffin-embedded, 4-5µm tissue section.

**STORAGE AND USAGE NOTES:** Store/Use each component according to the temperature and expiration on the label.

**PRECAUTIONS:** For use by laboratory professionals. See SDS for complete warnings, precautions, hazard and precautionary statements, and disposal information.

**CELLMATE WORKING SOLUTION PREPARATION:** Do not filter solution. Do not pipette solution onto slide or stain slide horizontally.

#	Action	Amount	Chemical/Reagent	Details
1	Add	20ml	CellMate-A™	Into a disposable slide jar.
2	Add	20ml	CellMate-B™	Mix thoroughly. Allow solution to sit for 30 minutes prior to use.

### STAINING PROCEDURE:

 Color coordinated steps denote stain baths that can be reused during autostainer configuration.

#	Action	With	Heat °C	Time		Details
				Mins	Secs	
1	Deparaffinize	Xylene or Substitute, 2 changes	--	5	--	5 minutes each change or as required if using a xylene substitute.
2	Rinse	Absolute Alcohol, 3 changes	--	1	--	1 minute each change or as required if using graded alcohols.
3	Rinse	Running Tap Water	--	1	--	
4	Immerse	CellMate™ Working Solution (Choose option 4a or 4b below).				
	4a	For Helicobacter pylori and bone marrow	--	10	--	Once complete, <b>rinse in running tap water (5 seconds)</b> and continue.
	4b	For mast cells	--	2-5	--	Once complete, <b>rinse in running tap water (5 seconds)</b> and continue.
5	Air Dry	On Benchtop, or in Empty Bath	--	5-10	--	Until section is completely dry.
6	Clear	Xylene or Substitute	--	--	1	5-10 dips for 1 second each.
7	Coverslip	Permanent Mounting Media	--	--	--	

1. Strumia: Laboratory and Clinical Medicine; pg 930
2. With modifications by AMTS R&D Department, 1979-2019.

**AUTOSTAINER CONFIGURATION AND NOTES:**

This stain kit in the pint size may be easily adapted for use on most open-platform autostainers using the staining procedure grid on the reverse side of this page. Standard CellMate Working Solution preparation procedures yields approximately 40ml of solution and must be scaled up to accommodate desired autostainer bath size. A minimum of 2 baths is required to perform this procedure excluding deparaffinization, hydration, dehydration, and clearing, or 8 baths to run the complete procedure.

**TEST YIELD:** \*Assumes pint kit and maximum slides per run. Actual Results may vary. S.C. denotes number of slides between "Solution Change".

Bath Type	Uses	Slides	S.C.	Bath Type	Uses	Slides	S.C.
20ml Plastic Slide Jar	50	1000	20	250ml Glass Stain Dish	4	750	188
30ml Glass Coplin Jar	33	990	30	200ml Bath Autostainer	5	800	160
40ml Hellendahl Jar	25	1000	40	400ml Bath Autostainer	2	680	340

**CE MARKINGS AND DESIGNATIONS:**

<b>REF</b>	Catalogue Number		Temperature Limitation		Manufacturer	American MasterTech Scientific 1330 Thurman St. Lodi, CA 95240 USA Tel 800 860 4073 Fax 209 368 4136
<b>LOT</b>	Batch Code		Use By	<b>EC REP</b>	Representative	Emergo Europe Prinsessegracht 20 2514 AP The Hague The Netherlands
<b>IVD</b>	In Vitro Diagnostic Medical Device		Consult Instructions Prior to Use	<b>CE</b>		
	Flammable		Toxic		Health Hazard	

**CONTACT INFORMATION:**

**American MasterTech Scientific**  
1330 Thurman St.  
Lodi, CA 95240 USA  
Tel 800 860 4073  
Fax 209 368 4136  
www.americanmastertech.com

**StatLab**  
2090 Commerce Drive  
McKinney, TX 75069 USA  
Tel 800 442 3573  
Fax 972 436 1369  
www.statlab.com

1. Strumia: Laboratory and Clinical Medicine; pg 930
2. With modifications by AMTS R&D Department, 1979-2019.

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