Human Skeletal Muscle

## Mallory's Trichrome Stain Kit Procedure

100ml Kit Item#: KTMAL      Liter Kit Item#: N/A  
 Pint Kit Item#: KTMALPT      Gallon Kit Item#: N/A

Control Slide(s)	Item#	Included Components
Heart	CSH0725P	Bouin's Fluid
Skeletal Muscle	CSS0725P	Acid Fuchsin Stain
Trichrome	CST0125P	Phosphomolybdic Acid
Uterus	CSU0325P	Aniline Blue / Orange G Stain

**PRINCIPLE AND RESULTS:** This kit is intended for use by laboratory professionals to stain routinely prepared paraffin embedded tissue specimens (in vitro) to identify collagen, elastic fibers, muscle, mucous, pituitary cells, and reticulum. Collagen fibers stain blue, reticulum blue, mucous shades of blue, elastic fibers pale pink, pale yellow or unstained, muscle red, alpha cells red, beta cells blue, and nuclei red.

**SPECIMEN CRITERIA:** Appropriately fixed, paraffin-embedded, 4-6µm tissue section.

**STORAGE AND USAGE NOTES:** Store/Use each component according to the temperature and expiration on the label.

**PRECAUTIONS:** For use by laboratory professionals. See SDS for complete warnings, precautions, hazard and precautionary statements, and disposal information.

**STAINING PROCEDURE:** Preheating required. See step 4 for more information.

 Color coordinated steps denote stain baths that can be reused during autostainer configuration.

#	Action	With	Heat °C	Time		Details
				Mins	Secs	
1	Deparaffinize	Xylene or Substitute, 2 changes	--	5	--	5 minutes each change or as required if using a xylene substitute.
2	Rinse	Absolute Alcohol, 3 changes	--	1	--	1 minute each change or as required if using graded alcohols.
3	Rinse	Running Tap Water	--	1	--	
4	Waterbath	Bouin's Fluid	56-62°	60	--	Immerse into preheated solution. (Optional: Immerse overnight at room temperature). Once complete, <b>rinse in running tap water (3 minutes)</b> until tissue is colorless and continue.
5	Immerse	Acid Fuchsin Stain	--	1-3	--	Without rinsing continue to next step.
6	Immerse	Phosphomolybdic Acid Solution	--	2	--	Without rinsing continue to next step.
7	Immerse	Aniline Blue / Orange G Stain	--	15	--	
8	Rinse	95% Reagent Alcohol	--	--	10-15	
9	Dehydrate	Absolute Alcohol, 3 changes	--	1	--	1 minute each change.
10	Clear	Xylene or Substitute, 3 changes	--	1	--	1 minute each change or as required if using a xylene substitute.
11	Coverslip	Permanent Mounting Media	--	--	--	

- Luna LG: Histopathologic Methods and Color Atlas of Special Stains and Tissue Artifacts; American Histolabs Inc; 1992, 439.
- Sheehan DC Hrapchak BB: Theory and Practice of Histotechnology, 1980, 190 – 191.
- With modifications by AMTS R&D Department, 1979-2019.

### AUTOSTAINER CONFIGURATION AND NOTES:

This stain kit in the pint or larger size may be easily adapted for use on most open-platform autostainers using the staining procedure grid on the reverse side of this page. A minimum of 5 baths is required to perform this procedure excluding deparaffinization, hydration, dehydration, and clearing, or 16 baths to run the complete procedure.

**TEST YIELD:** \*Assumes pint kit and maximum slides per run. Actual Results may vary. S.C. denotes number of slides between "Solution Change".

Bath Type	Uses	Slides	S.C.	Bath Type	Uses	Slides	S.C.
20ml Plastic Slide Jar	25	500	20	250ml Glass Stain Dish	2	375	188
30ml Glass Coplin Jar	16	480	30	200ml Bath Autostainer	2	320	160
40ml Hellendahl Jar	12	480	40	400ml Bath Autostainer	1	340	340

### CE MARKINGS AND DESIGNATIONS:

<b>REF</b>	Catalogue Number		Temperature Limitation		Manufacturer	American MasterTech Scientific 1330 Thurman St. Lodi, CA 95240 USA Tel 800 860 4073 Fax 209 368 4136
<b>LOT</b>	Batch Code		Use By	<b>EC REP</b>	Representative	Emergo Europe Prinsessegracht 20 2514 AP The Hague The Netherlands
<b>IVD</b>	In Vitro Diagnostic Medical Device		Consult Instructions Prior to Use	<b>CE</b>		
 GHS05	Corrosive	 GHS07	Irritant	 GHS08	Health Hazard	

### CONTACT INFORMATION:

**American MasterTech Scientific**  
1330 Thurman St.  
Lodi, CA 95240 USA  
Tel 800 860 4073  
Fax 209 368 4136  
www.americanmastertech.com

**StatLab**  
2090 Commerce Drive  
McKinney, TX 75069 USA  
Tel 800 442 3573  
Fax 972 436 1369  
www.statlab.com

1. Luna LG: Histopathologic Methods and Color Atlas of Special Stains and Tissue Artifacts; American Histolabs Inc; 1992, 439.
2. Sheehan DC Hrapchak BB: Theory and Practice of Histotechnology, 1980, 190 – 191.
3. With modifications by AMTS R&D Department, 1979-2019.

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**COMPOSANTS INCLUS:** Bouin's Fluid, Acid Fuchsin Stain, Phosphomolybdic Acid, Aniline Blue/Orange G Stain.

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**COMPONENTES INCLUIDOS:** Bouin's Fluid, Acid Fuchsin Stain, Phosphomolybdic Acid, Aniline Blue/Orange G Stain.

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