Human Skin

Melanin Bleach Kit Procedure

100ml Kit Item #: KTMBK

Liter Kit Item#: N/A

Pint Kit Item #: N/A

Gallon Kit Item#: N/A

Control Slide(s) | Item#

Skin | CSS0825P

Included Components

1% Potassium Permanganate

2% Oxalic Acid

PRINCIPLE AND RESULTS: This kit is intended for use by laboratory professionals to remove the melanin pigment (in vitro) from hair, skin, retina or substantia nigra of the brain.

SPECIMEN CRITERIA: Appropriately fixed, paraffin-embedded, 4-5µm tissue section.

STORAGE AND USAGE NOTES: Store/Use each component according to the temperature and expiration on the label.

PRECAUTIONS: For use by laboratory professionals. See SDS for complete warnings, precautions, hazard and precautionary statements, and disposal information.

STAINING PROCEDURE:

Color coordinated steps denote stain baths that can be reused during autostainer configuration.

#	Action	With	Heat °C	Time		Details
				Mins	Secs	
1	Deparaffinize	Xylene or Substitute, 2 changes	--	5	--	5 minutes each change or as required if using a xylene substitute.
2	Rinse	Absolute Alcohol, 3 changes	--	1	--	1 minute each change or as required if using graded alcohols.
3	Rinse	Running DI Water	--	1	--	
4	Immerse	1% Potassium Permanganate	--	20-240	--	Adjust time depending on tissue type. Once complete, rinse in running DI water (1 minute) and continue.
5	Immerse	2% Oxalic Acid	--	1-2	--	Until tissue is colorless. Once complete, rinse in running DI water (1 minute) . Perform special or IHC stain per instructions excluding deparaffinization and hydration steps.

- Lillie R: Histopathologic Technique and Practical Histochemistry 3rd Edition, 1965
- With modifications by AMTS R&D Department, 1979-2019.

AUTOSTAINER CONFIGURATION AND NOTES:

This stain kit is not available in the pint or larger size.

TEST YIELD: *Assumes pint kit and maximum slides per run. Actual Results may vary. S.C. denotes number of slides between "Solution Change".

Bath Type	Uses	Slides	S.C.	Bath Type	Uses	Slides	S.C.
20ml Plastic Slide Jar	5	10	20	250ml Glass Stain Dish	n/a	n/a	n/a
30ml Glass Coplin Jar	3	90	30	200ml Bath Autostainer	n/a	n/a	n/a
40ml Hellendahl Jar	2	80	40	400ml Bath Autostainer	n/a	n/a	n/a

CE MARKINGS AND DESIGNATIONS:

REF	Catalogue Number		Temperature Limitation		Manufacturer	American MasterTech Scientific 1330 Thurman St. Lodi, CA 95240 USA Tel 800 860 4073 Fax 209 368 4136
LOT	Batch Code		Use By	EC REP	Representative	Emergo Europe Prinsessegracht 20 2514 AP The Hague The Netherlands
IVD	In Vitro Diagnostic Medical Device		Consult Instructions Prior to Use	CE		

CONTACT INFORMATION:

American MasterTech Scientific
1330 Thurman St.
Lodi, CA 95240 USA
Tel 800 860 4073
Fax 209 368 4136
www.americanmastertech.com

StatLab
2090 Commerce Drive
McKinney, TX 75069 USA
Tel 800 442 3573
Fax 972 436 1369
www.statlab.com

1. Lillie R: Histopathologic Technique and Practical Histochemistry 3rd Edition, 1965
2. With modifications by AMTS R&D Department, 1979-2019.

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COMPOSANTS INCLUS: 1% Potassium Permanganate, 2% Oxalic Acid

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