Human Gastric Tissue

Genta Triple Stain Kit Procedure, Modified

100ml Kit Item #: KTMGT

Liter Kit Item#: N/A

Pint Kit Item #: N/A

Gallon Kit Item#: N/A

Control Slide(s)

Item#

Helicobacter pylori CSH0125P

Included Components

Hydroquinone Capsules	0.1% Silver Nitrate
2.5% Gum Mastic	Alcian Blue pH 2.5 Stain
1% Silver Nitrate	Modified Mayer's Hematoxylin
Oxidizer Solution	Eosin Y Stain
10% Zinc Formalin	

PRINCIPLE AND RESULTS: This kit is intended for use by laboratory professionals to stain routinely prepared paraffin embedded tissue specimens (in vitro) to identify *Helicobacter pylori*, goblet cells, acid mucins, mucous producing cells, and oxyntic cells. *Helicobacter pylori* and nuclei stain dark brown to black, goblet cells and acid mucins shades of blue, mucous cells pink to yellow, oxyntic (acid) producing cells light yellow, and background shades of pink.


SPECIMEN CRITERIA: Appropriately fixed, paraffin-embedded, 4µm tissue section.

PRECAUTIONS: For use by laboratory professionals. See SDS for complete warnings, precautions, hazard and precautionary statements, and disposal information.

REDUCING SOLUTION PREPARATION: Prepare solution at time of use. Solution expires after one use. Reducing Silver Solution reacts with metals. Do not use metal forceps, racks, or tools with this solution.

#	Action	Amount	Chemical/Reagent	Details
1	Add	25ml	Distilled/DI Water	Into a chemically clean container or new/unused plasticware.
2	Add	1	Hydroquinone Capsule	Empty contents of capsule and mix until completely dissolved.
3	Add	15ml	2.5% Gum Mastic	Mix thoroughly.
4	Filter	--	Medium Filter Paper	Into a chemically clean container or new/unused plasticware.
5	Add	6 Drops	1% Silver Nitrate	Mix thoroughly.


STAINING PROCEDURE: Preheating Required. See steps 6 and 10 for more information. Microwave procedure located on opposite page.

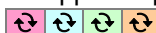 Color coordinated steps denote stain baths that can be reused during autostainer configuration.

#	Action	With	Heat °C	Time		Details
				Mins	Secs	
1	Deparaffinize	Xylene or Substitute, 2 changes	--	5	--	5 minutes each change or as required if using a xylene substitute.
2	Rinse	Absolute Alcohol, 3 changes	--	1	--	1 minute each change or as required if using graded alcohols.
3	Immerse	Oxidizer Solution	--	20	--	
4	Rinse	Running DI Water	--	1	--	
5	Immerse	10% Zinc Formalin	--	5	--	Once complete, rinse in running DI water (1 minute) and continue.
6	Waterbath	0.1% Silver Nitrate	70°	5	--	Immerse into preheated solution. Once complete, rinse in running DI water (1 minute) and continue. Discard solution after use.
7	Rinse	Absolute Alcohol, 2 changes	--	1	--	1 minute each change.
8	Immerse	2.5% Gum Mastic	--	3	--	During this step, prepare the reducing solution.
9	Air Dry	On Benchtop, or in Empty Bath	--	1-2	--	Until gum mastic is completely dry.
10	Waterbath	Reducing Solution	70°	10-15	--	Immerse into a preheated solution until section is tan to brown.
11	Rinse	Absolute Alcohol, 3 changes	--	1	--	1 minute each change. Once complete, rinse in running DI water (1 minute), and continue.
12	Immerse	Alcian Blue pH 2.5	--	10-20	--	Once complete, rinse in running DI water (1 minute) and continue.
13	Immerse	Modified Mayer's Hematoxylin	--	5	--	
14	Rinse	Running Tap Water	--	2	--	
15	Immerse	Eosin Y Stain	--	5	--	
16	Dehydrate	Absolute Alcohol, 3 changes	--	1	--	1 minute each change.
17	Clear	Xylene or Substitute, 3 changes	--	1	--	1 minute each change or as required if using a xylene substitute.
18	Coverslip	Permanent Mounting Media	--	--	--	Coverslip immediately.

1. Genta RM, Robason GO, Graham DY: Simultaneous visualization of *Helicobacter pylori* and gastric morphology, Hum Pathol 25:221 – 226.
2. With modifications by AMTS R&D Department, 1979-2018.

MICROWAVE STAINING PROCEDURE:

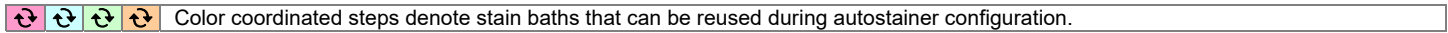#	Action	With	Heat °C	Time		Details
				Mins	Secs	
1	Deparaffinize	Xylene or Substitute, 2 changes	--	5	--	5 minutes each change or as required if using a xylene substitute.
2	Rinse	Absolute Alcohol, 3 changes	--	1	--	1 minute each change or as required if using graded alcohols.
3	Immerse	Oxidizer Solution	--	20	--	
4	Rinse	Running DI Water	--	1	--	
5	Immerse	10% Zinc Formalin	--	5	--	Once complete, rinse in running DI water (1 minute) and continue.
6	Microwave	0.1% Silver Nitrate	80-90°	--	--	Heat 0.1% Silver Nitrate in a loosely capped slide jar (without slide). Do not boil. Remove from microwave, tighten cap, and agitate. Immerse slide and incubate for 2 minutes while agitating periodically. Once complete, rinse in running DI water (1 minute) and continue.
7	Rinse	Absolute Alcohol, 2 changes	--	1	--	1 minute each change.
8	Immerse	2.5% Gum Mastic	--	3	--	During this step, prepare the reducing solution.
9	Air Dry	On Benchtop, or in Empty Bath	--	1-2	--	Until gum mastic is completely dry.
10	Microwave	Reducing Solution	80-90°	--	--	Heat Reducing Solution in a loosely capped slide (without slide). Do not boil. Remove from microwave, tighten cap, and agitate. Immerse slide, agitate thoroughly, and incubate until section is brown. Repeat heating/incubating step as needed. Discard solution after use.
11	Rinse	Absolute Alcohol, 3 changes	--	1	--	1 minute each change. Once complete, rinse in running DI water (1 minute) and continue.
12	Immerse	Alcian Blue pH 2.5	--	10-20	--	Once complete, rinse in running DI water (1 minute) and continue.
13	Immerse	Modified Mayer's Hematoxylin	--	5	--	
14	Rinse	Running Tap Water	--	2	--	
15	Immerse	Eosin Y Stain	--	5	--	
16	Dehydrate	Absolute Alcohol, 3 changes	--	1	--	1 minute each change.
17	Clear	Xylene/Substitute, 3 changes	--	1	--	1 minute each change or as required if using a xylene substitute.
18	Coverslip	Permanent Mounting Media	--	--	--	Coverslip immediately.

AUTOSTAINER CONFIGURATION AND NOTES: This stain kit is not available in the pint size or larger.

TEST YIELD: *Assumes pint kit and maximum slides per run. Actual Results may vary. S.C. denotes number of slides between "Solution Change".

Bath Type	Uses	Slides	S.C.	Bath Type	Uses	Slides	S.C.
20ml Plastic Slide Jar	5	20	4	40ml Hellendahl Jar	2	32	16
30ml Glass Coplin Jar	3	24	8				

CE MARKINGS AND DESIGNATIONS:

REF	Catalogue Number		Temperature Limitation		Manufacturer	American MasterTech Scientific 1330 Thurman St. Lodi, CA 95240 USA Tel 800 860 4073 Fax 209 368 4136
LOT	Batch Code		Use By	EC REP	Representative	Emergo Europe Prinsessegracht 20 2514 AP The Hague The Netherlands
IVD	In Vitro Diagnostic Medical Device		Consult Instructions Prior to Use	CE		
	Flammable		Corrosive		Irritant	
GHS02		GHS05		GHS07		
	Health Hazard		Environmentally Damaging			
GHS08		GHS09				

CONTACT INFORMATION:

American MasterTech Scientific
1330 Thurman St. • Lodi, CA 95240 USA
Tel 800 860 4073 • Fax 209 368 4136
www.americanmastertech.com

StatLab
2090 Commerce Drive • McKinney, TX 75069 USA
Tel 800 442 3573 • Fax 972 436 1369
www.statlab.com

- Genta RM, Robason GO, Graham DY: Simultaneous visualization of Helicobacter pylori and gastric morphology, Hum Pathol 25:221 – 226.
- With modifications by AMTS R&D Department, 1979-2018.

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COMPOSANTS INCLUS: Hydroquinone Capsules, 2.5% Gum Mastic, 1% Silver Nitrate, Oxidizer Solution, 10% Zinc Formalin, 0.1% Silver Nitrate, Alcian Blue pH 2.5 Stain, Modified Mayer's Hematoxylin, Eosin Y Stain.

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- Genta RM, Robason GO, Graham DY: Simultaneous visualization of *Helicobacter pylori* and gastric morphology, Hum Pathol 25:221 – 226.
- With modifications by AMTS R&D Department, 1979-2018.

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1. Genta RM, Robason GO, Graham DY: Simultaneous visualization of Helicobacter pylori and gastric morphology, Hum Pathol 25:221 – 226.
2. With modifications by AMTS R&D Department, 1979-2018.

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COMPONENTES INCLUIDOS: Hydroquinone Capsules, 2.5% Gum Mastic, 1% Silver Nitrate, Oxidizer Solution, 10% Zinc Formalin, 0.1% Silver Nitrate, Alcian Blue pH 2.5 Stain, Modified Mayer's Hematoxylin, Eosin Y Stain.

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