Human Ovary Tumor

Microwave Modified Grimelius Stain Kit Procedure

100ml Kit Item #: KTMMG

Liter Kit Item#: N/A

Pint Kit Item #: N/A

Gallon Kit Item#: N/A

Control Slide(s) | **Item#**
Argentaffin-Argyrophil | CSA0525P

Included Components

Sodium Sulfite	1% Silver Nitrate Solution
Hydroquinone	Nuclear Fast Red Stain

PRINCIPLE AND RESULTS: This kit is intended for use by laboratory professionals to stain routinely prepared paraffin embedded tissue specimens (in vitro) to identify argyrophil cells. Argyrophil cells stain black and background shades of yellow to pink.

SPECIMEN CRITERIA: Appropriately fixed, paraffin-embedded, 4-5µm tissue section.

STORAGE AND USAGE NOTES: Requires the use of a strong tissue adhesive. Store/Use each component according to the temperature and expiration on the label.

PRECAUTIONS: For use by laboratory professionals. See SDS for complete warnings, precautions, hazard and precautionary statements, and disposal information.

BODIAN'S REDUCING SOLUTION PREPARATION: Prepare solution at time of use.

#	Action	Amount	Chemical/Reagent	Details
1	Add	25ml	Distilled/DI Water	Into a chemically cleaned container or new/unused plasticware.
2	Add	1.25gm	Sodium Sulfite	Mix thoroughly.
3	Add	0.25gm	Hydroquinone	Mix thoroughly.

STAINING PROCEDURE:

 Color coordinated steps denote stain baths that can be reused during autostainer configuration.

#	Action	With	Heat °C	Time		Details
				Mins	Secs	
1	Deparaffinize	Xylene or Substitute, 2 changes	--	5	--	5 minutes each change or as required if using a xylene substitute.
2	Rinse	Absolute Alcohol, 3 changes	--	1	--	1 minute each change or as required if using graded alcohols.
3	Rinse	Running DI Water	--	1	--	
4	Microwave	1% Silver Nitrate Solution	60-70°	--	--	Immerse slide into a slide jar containing 20ml of 1% Silver Nitrate Solution, loosely cap jar, and heat. Save this solution! Once complete, rinse in running DI water (30 seconds) and continue.
5	Microwave	Bodian's Reducing Solution	60-70°	--	--	Heat 25ml of Bodian's Reducing Solution (without slide). Remove from microwave and immerse slide into the solution for 3-5 minutes. Save this solution! Once complete, rinse in running DI water (1 minute) and continue.
6	Immerse	1% Silver Nitrate Solution (From step 4)	--	1	--	One complete, rinse in running DI water (1 minute) and continue. Discard solution; do not reuse.
7	Immerse	Bodian's Reducing Solution (From step 5)	--	2	--	Once complete, rinse in running DI water (5 minutes) and continue. Discard solution; do not reuse.
8	Immerse	Nuclear Fast Red Stain	--	3	--	One complete, rinse in running DI water (1 minute) and continue.
9	Dehydrate	Absolute Alcohol, 3 changes	--	1	--	1 minute each change.
10	Clear	Xylene or Substitute, 3 changes	--	1	--	1 minute each change or as required if using a xylene substitute.
11	Coverslip	Permanent Mounting Media	--	--	--	

1. Pascual JS: A new method for easy demonstration of argyrophil cells; StainTechnol, 51: 1976, 231-235.
2. With modifications by AMTS R&D Department, 1979-2019.

AUTOSTAINER CONFIGURATION AND NOTES:

This stain kit is not available in the pint or larger size.

TEST YIELD: *Assumes pint kit and maximum slides per run. Actual Results may vary. S.C. denotes number of slides between "Solution Change".

Bath Type	Uses	Slides	S.C.	Bath Type	Uses	Slides	S.C.
20ml Plastic Slide Jar	5	20	4	250ml Glass Stain Dish	n/a	n/a	n/a
30ml Glass Coplin Jar	3	24	8	200ml Bath Autostainer	n/a	n/a	n/a
40ml Hellendahl Jar	2	32	16	400ml Bath Autostainer	n/a	n/a	n/a

CE MARKINGS AND DESIGNATIONS:

REF	Catalogue Number		Temperature Limitation		Manufacturer	American MasterTech Scientific 1330 Thurman St. Lodi, CA 95240 USA Tel 800 860 4073 Fax 209 368 4136
LOT	Batch Code		Use By	EC REP	Representative	Emergo Europe Prinsessegracht 20 2514 AP The Hague The Netherlands
IVD	In Vitro Diagnostic Medical Device		Consult Instructions Prior to Use		Corrosive	CE
	Irritant		Health Hazard		Environmentally Damaging	
GHS07		GHS08		GHS09		

CONTACT INFORMATION:

American MasterTech Scientific
1330 Thurman St.
Lodi, CA 95240 USA
Tel 800 860 4073
Fax 209 368 4136
www.americanmastertech.com

StatLab
2090 Commerce Drive
McKinney, TX 75069 USA
Tel 800 442 3573
Fax 972 436 1369
www.statlab.com

1. Pascual JS: A new method for easy demonstration of argyrophil cells; StainTechnol, 51: 1976, 231-235.
2. With modifications by AMTS R&D Department, 1979-2019.

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COMPOSANTS INCLUS: Sodium Sulfite, Hydroquinone, 1% Silver Nitrate Solution, Nuclear Fast Red Stain

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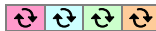
- Pascual JS: A new method for easy demonstration of argyrophil cells; StainTechnol, 51: 1976, 231-235.
- With modifications by AMTS R&D Department, 1979-2019.

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COMPONENTES INCLUIDOS: Sodium Sulfite, Hydroquinone, 1% Silver Nitrate Solution, Nuclear Fast Red Stain

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