Acid Fast Bacteria

## Modified Truant's Fluorescent Acid Fast Bacteria (A.F.B.) Stain Kit Procedure

**100ml Kit Item #:** KTMTF

**Liter Kit Item#:** N/A

**Pint Kit Item #:** N/A

**Gallon Kit Item#:** N/A

Control Slide(s)	Item#
Acid Fast Bacteria	CSA0125P

Included Components
Auramine O-Rhodamine B Stain
Acid Alcohol Solution
Eriochrome Black T Stain

**PRINCIPLE AND RESULTS:** This kit is intended for use by laboratory professionals to stain routinely prepared paraffin embedded tissue specimens (in vitro) to identify acid fast bacteria. Acid fast bacteria stain fluorescent orange to yellow (use a fluorescent microscope with a 450-490nm excitation filter and a 510nm barrier filter).

**SPECIMEN CRITERIA:** Appropriately fixed, paraffin-embedded, 4-5µm tissue section or 6-8µm frozen section.

**STORAGE AND USAGE NOTES:** Store/Use each component according to the temperature and expiration on the label.

**PRECAUTIONS:** For use by laboratory professionals. See SDS for complete warnings, precautions, hazard and precautionary statements, and disposal information.

**STAINING PROCEDURE:** Preheating may be required. See step 4a for more information.

 Color coordinated steps denote stain baths that can be reused during autostainer configuration.

#	Action	With	Heat °C	Time		Details
				Mins	Secs	
1	Deparaffinize	Xylene or Substitute, 2 changes	--	5	--	5 minutes each change or as required if using a xylene substitute.
2	Rinse	Absolute Alcohol, 3 changes	--	1	--	1 minute each change or as required if using graded alcohols.
3	Rinse	Running DI Water	--	1	--	
4	Choose routine waterbath (4a) or microwave (4b) procedure below.					
4a	Waterbath	Auramine O-Rhodamine B Stain	60°	20-30	--	Immerse into 20ml of preheated solution. Agitate periodically. Once complete, <b>rinse in running DI water (1 minute)</b> and continue at step 5.
4b	Microwave	Auramine O-Rhodamine B Stain	80°	--	--	Heat a slide jar containing Auramine O-Rhodamine B Stain (without slides). Immerse slide into heated solution and incubate for 5 minutes, while agitating. Once complete, <b>rinse in running DI water (1 minute)</b> and continue at step 5.
5	Immerse	Acid Alcohol Solution, 2 changes	--	--	90	90 seconds each change. Once complete, <b>rinse in running DI water (1 minute)</b> and continue.
6	Immerse	Eriochrome Black T Stain	--	--	8-10	Once complete, <b>rinse in running DI water (1 minute)</b> and continue.
7	Air Dry	On Benchtop, or in Empty Bath	--	5-10	--	Until section is completely dry.
8	Clear	Xylene or Substitute	--	--	1	5-10 dips for 1 second each.
9	Coverslip	Permanent Mounting Media	--	--	--	

- Matthaei E: Simplified Fluorescence microscopy of tubercle bacilli; J Gen Microbiol; 1950, 4: 393-398.
- Truant JP Brett WA Thomas W: Fluorescence microscopy of tubercle bacilli stained with Auramine and Rhodamine; Henry Ford Hosp Med Bul; 10: 1962, 287-296.
- Churukian CJ: J of Histo; 14: 1991, 117.
- With modifications by AMTS R&D Department, 1979-2019.

### AUTOSTAINER CONFIGURATION AND NOTES:

This stain kit is not available in the pint or larger size.

**TEST YIELD:** \*Assumes pint kit and maximum slides per run. Actual Results may vary. S.C. denotes number of slides between "Solution Change".

Bath Type	Uses	Slides	S.C.	Bath Type	Uses	Slides	S.C.
20ml Plastic Slide Jar	5	45	9	250ml Glass Stain Dish	n/a	n/a	n/a
30ml Glass Coplin Jar	3	41	14	200ml Bath Autostainer	n/a	n/a	n/a
40ml Hellendahl Jar	2	36	18	400ml Bath Autostainer	n/a	n/a	n/a

### CE MARKINGS AND DESIGNATIONS:

<b>REF</b>	Catalogue Number		Temperature Limitation		Manufacturer	American MasterTech Scientific 1330 Thurman St. Lodi, CA 95240 USA Tel 800 860 4073 Fax 209 368 4136
<b>LOT</b>	Batch Code		Use By	<b>EC REP</b>	Representative	Emergo Europe Prinsessegracht 20 2514 AP The Hague The Netherlands
<b>IVD</b>	In Vitro Diagnostic Medical Device		Consult Instructions Prior to Use	<b>CE</b>		
	Flammable		Corrosive		Health Hazard	
GHS02		GHS05		GHS08		

### CONTACT INFORMATION:

**American MasterTech Scientific**  
1330 Thurman St.  
Lodi, CA 95240 USA  
Tel 800 860 4073  
Fax 209 368 4136  
www.americanmastertech.com

**StatLab**  
2090 Commerce Drive  
McKinney, TX 75069 USA  
Tel 800 442 3573  
Fax 972 436 1369  
www.statlab.com

1. Matthaei E: Simplified Fluorescence microscopy of tubercle bacilli; J Gen Microbiol; 1950, 4: 393-398.
2. Truant JP Brett WA Thomas W: Fluorescence microscopy of tubercle bacilli stained with Auramine and Rhodamine; Henry Ford Hosp Med Bul; 10: 1962, 287-296.
3. Churukian CJ: J of Histo; 14: 1991, 117.
4. With modifications by AMTS R&D Department, 1979-2019.

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**COMPOSANTS INCLUS:** Auramine O-Rhodamine B Stain, Acid Alcohol Solution, Eriochrome Black T Stain.

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