American MasterTech  
scientific laboratory supplies

# ClearMount™

## Permanent Mounting Medium

(Rev. 3/8/18)

### **ClearMount™, The product and its purpose:**

Developed using the finest acrylic copolymers, ClearMount™ is the most advanced non-aqueous mounting medium available. The highly refined acrylic copolymers used in ClearMount™ offer excellent color retention, refractivity, and do not yellow with age during storage. Unlike most synthetic mounting mediums which limit their application to specific primary clearing agents such as Xylene, ClearMount™ offers compatibility with virtually any clearing agent. With ClearMount™, microscope slide and coverslip will set in five minutes at room temperature allowing immediate microscopic examination. ClearMount™ may be the fastest drying, most advanced mounting medium in the world. ClearMount™ is intended for In Vitro Diagnostic use. Active ingredients include; Xylene, Toluene, and Acrylic Copolymer.

### **ClearMount™, Instructions for use:**

When working with traditional clearing agents such as Xylene or Toluene, use ClearMount™ as you would any routine mounting medium. If you are using a Xylene substitute, a few minor procedure changes will assure quality results time after time. Always use four changes of absolute alcohol after the last stain in a staining procedure (Example: Eosin stain or EA\*50). Xylene alternatives are less tolerant of water carry-over. Employing four changes of absolute alcohol virtually eliminates water carry-over. Alcohols should be rotated often. Make sure the alcohol you use does not contain high levels of methanol. Use three changes of clearing agent when clearing slides before coverslipping. Agitate slides and immerse in each change of clearing agent for five minutes. Do not let slides dry out before coverslipping! Use a sufficient amount of ClearMount™ to provide thorough coverage and eliminate air bubbles after drying. To remove an applied coverslip, soak slide in Xylene until coverslip detaches, then apply a new coverslip using ClearMount™. If necessary, ClearMount™ can be thinned with Xylene. NOTE: Picric acid stains may bleed or fade.

Note: Test ClearMount™ compatibility when using a Xylene substitute. To test compatibility, add 1 to 2 ml. of clearing agent to a small glass test tube. Add 1 to 3 drops of ClearMount™ to the test tube and agitate test tube until liquid is well blended. Set test tube aside for 1 hour; if solution remains transparent, the clearing agent is compatible with ClearMount™.

### **ClearMount™, Storage and stability:**

Store ClearMount™ at 15- 30°C; it is stable under normal laboratory conditions. Keep tightly sealed when not in use. See product label for shelf life.

### **ClearMount™, Precautions:**

See Safety Data Sheet (SDS) for warnings and precautions.

### **ClearMount™, Disposal:**

Dispose of ClearMount™ in accordance with all local, state, federal, environmental and/or EU regulations.

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### CE MARKINGS AND DESIGNATIONS:

<b>REF</b>	Catalogue Number		Temperature Limitation		Manufacturer	American MasterTech Scientific 1330 Thurman St. Lodi, CA 95240 USA Tel 800 860 4073 Fax 209 368 4136
<b>LOT</b>	Batch Code		Use By		Representative	Emergo Europe Prinsesegracht 20 2514 AP The Hague The Netherlands
<b>IVD</b>	In Vitro Diagnostic Medical Device		Consult Instructions Prior to Use	 GHS02	Flammable	
 GHS07	Irritant	 GHS08	Health Hazard			

Copyright © 2015-2018 American MasterTech Scientific. All rights reserved.

No part of these pages may be used for any purpose other than personal use. Therefore, reproduction, modification, storage in a retrieval system or retransmission, in any form or by any means, for reasons other than personal use, is strictly prohibited without prior written permission.